**Lateral Flow Immunoassay to detect
IgG antibodies against PF4/polyanion-complexes
English: Page 1-6**

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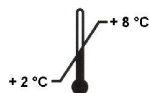
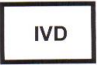
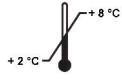
For further languages please visit: www.milenia-biotec.de or send an email to: info@milenia-biotec.de

Warnings and Precautions

- Read **these instructions** carefully before performing the test. Follow them step by step!
- Store reagents at 2 - 8° C!
- Do not use the kit after expiration date!
- Do not use test unit if foil pouch is damaged!
- Protect test units from moisture!
- Do not interchange components of packages of different batch codes!
- All samples, used units, pipettes, tips, and other contaminated material should be handled carefully and as potentially infectious.
- For professional users.

Materials Supplied, Storage and Stability

REF: MQHIT 1					
Component	Cat.-No.	Content	Preparation	Storage at	Shelf life
HIT Test Unit; membrane coated with polyclonal (goat) anti-human IgG antibodies, anti-ligand antibodies conjugated to gold particles	MQSHIT	20	ready to use	2 - 8 °C Use immediately after opening the foil pouch!	in the <u>closed</u> foil pouch until expiration date
Sample Buffer, Buffer, pH 7.4	MQBHIT	1 vial à 2 mL (white)	ready to use	2 - 8 °C	until expiration date
MQHIT 1: This bag contains 20 Evaluation Cards					

REF: MQHIT Z					
Component	Cat.-No.	Content	Preparation	Storage at	Shelf life
HIT Test Unit; membrane coated with polyclonal (goat) anti-human IgG antibodies, anti-ligand antibodies conjugated to gold particles	MQSHIT	5	ready to use	2 - 8 °C Use immediately after opening the foil pouch!	in the <u>closed</u> foil pouch until expiration date
Sample Buffer, Buffer, pH 7.4	MQBHITZ	1 vial à 0.5 mL (white)	ready to use	2 - 8 °C	until expiration date
MQHIT Z: This bag contains 5 Evaluation Cards					

Material Safety Data Sheets are available on request (look as well www.milenia-biotec.de).

Intended Use

- For *in vitro* diagnostic use.
- Milenia® QuickLine HIT is designed for qualitative detection of IgG antibodies against PF4/polyanion-complexes.

Materials Required But Not Supplied

Pipet and tips: 5 µL (serum)

Specimen Collection and Preparation

Serum
Avoid hemolysis; separate serum from blood cells within two hours

Volume Required

Serum: 5 µL

Summary and Explanation

Heparin induced thrombocytopenia (HIT) is a life-threatening disease associated with exposure to unfractionated or (less commonly) low-molecular-weight heparin, drugs which are used very frequently for various indications. HIT II occurs in up to 5 % of patients on heparin. This prevalence varies with patient population and the nature of the disease and the type of heparin, thus the anticoagulant. Cardiac and major orthopedic surgeries expose a higher risk than other clinical situations when heparin is used.

HIT is caused by antibodies directed against a complex of platelet factor 4 (PF4) and polyanions such as heparin. The pathogenic and thrombogenic species is immunoglobulin IgG. There is evidence for a correlation between antibody concentration and clinical severity. Platelet activation and massively enhanced thrombin generation promote venous and/ or arterial thromboembolism. The rate of severe patient injury (e.g. amputations) and mortality is very high.

Patients with HIT II require alternative anticoagulation, but the drugs used in this setting have a high rate of hemorrhagic complications. There is no antidote available. Monitoring with standard methods is either not reliable or impossible. On top of this the drug costs are significantly higher than for heparins. Alternative anticoagulants should be reserved for patients with high clinical probability for HIT and with confirmed antibodies. Excluding HIT-antibodies is an important step towards selecting the most appropriate anticoagulation strategy.

Method and Test Principle

The Milenia[®] QuickLine HIT is a lateral flow immunoassay designed for qualitative determination of antibodies against PF4/polyanion-complexes in human serum.

The sample is pipetted in the sample application port of the test unit and the sample buffer is added immediately. The buffer forces the sample to migrate through the membrane of the test unit.

Immobilized anti-human IgG on the nitrocellulose membrane binds patient's IgG-antibodies which are previously captured by the PF4/polyanion-complex which is detected by intensely colored gold nanoparticles.

The presence of PF4/polyanion-complexes becomes visible at a colored test line (T). This test line - if it is positive - has to be the same or a darker shade of color than the given one on the provided evaluation card.

The surplus of gold particles continues to migrate through the membrane and is captured at the control line (C) by specific antibodies. A well visible line appears after incubation time.

References

1. Greinacher A. Heparin-induced thrombocytopenia. *J Thromb Haemost.* 2009; 7 Suppl 1:9-12. Review.
2. Wilke T, Tesch S, Scholz A, Kohlmann T, Greinacher A. The costs of heparin-induced thrombocytopenia: a patient-based cost of illness analysis. *J Thromb Haemost.* 2009;7(5):766-73
3. Bakchoul T, Giptner A, Najaoui A, Bein G, Santoso S, Sachs UJ. Prospective evaluation of PF4/heparin immunoassays for the diagnosis of heparin-induced thrombocytopenia. *J Thromb Haemost.* 2009;7 (8): 1260-5.
4. Sachs UJ, Manuscript in preparation
5. Warkentin TE, Greinacher A, Koster A, Lincoff AM; American College of Chest Physicians. Treatment and prevention of heparin-induced thrombocytopenia: American College of Chest Physicians Evidence-Based Clinical Practice Guidelines (8th Edition). *Chest.* 2008 Jun;133(6 Suppl):340S-380S.

Test Performance

Important notes:

- Do not interchange components of different lots!
- The test unit is sensitive to humidity. In case of damaged package do not use test unit!
- Before opening bring the test unit and sample buffer to room temperature (18 – 28 °C).
- Use the test unit immediately after opening. Do not store at room temperature!

	Step	
1.		Pipet 5 µL serum into the sample application port of the test unit.
2.		Add 2 drops sample buffer Note: Do not contaminate the vial with sample material!
3.	Incubation	10 minutes at room temperature (18 - 28°C).
4.	Evaluation (visual)	The result will be evaluated visually by the provided evaluation card: The test is positive if the test line shows the same or a darker shade of color after incubation time than the given line in the evaluation card (s. results interpretation and functional control) The result must be interpreted immediately after incubation time.

Results Interpretation

- **Milenia® QuickLine HIT- Test Line- positive:** An intensely colored line appears on the membrane.
- **Milenia® QuickLine HIT- Test Line- negative:** The test line remains uncolored.
- **Milenia® QuickLine HIT- Control Line- positiv:** The control-line (C) must be colored (s. Function Control).

The interpretation of Milenia® QuickLine HIT can only be evaluated **visually** by the provided **evaluation card**.

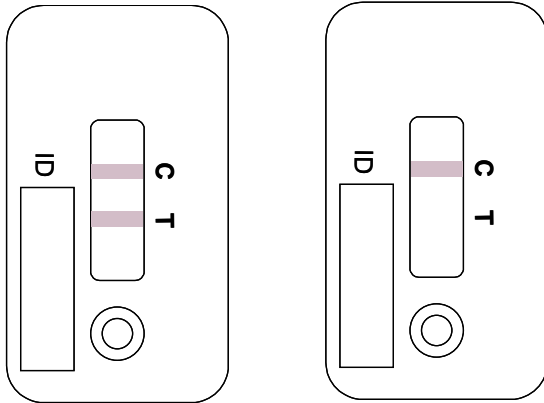
Note: Results should be verified concerning the entire clinical status of the patient. Additionally every decision for therapy should be taken individually.

Function Control

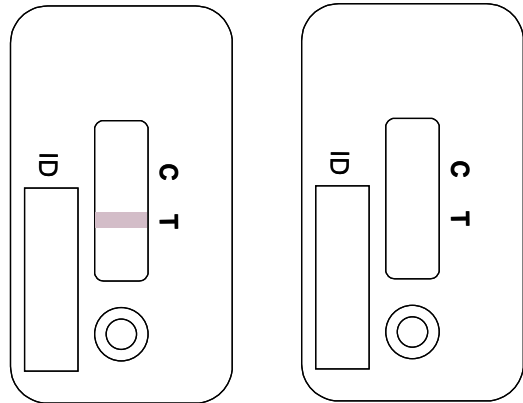
In any case, the Control line (C) has to appear!

It is only used as functional control and can not be used for the interpretation of the test line.
If the control line is not detectable after 10 minutes incubation time, the result is invalid and the analysis has to be repeated with a new test unit!

Valid results

Invalid results

Assay Characteristics

Sample material:	5 μ L serum
Incubation time:	10 minutes at room temperature (18 – 28 °C)
Sensitivity:	100%
Specificity:	93%

Limitation / Interferences

Lipemia, bilirubin, and haemolysis do not show any interference influence on the results.

Lateral Flow Immunoassay to detect
IgG antibodies against PF4/polyanion-complexes
English: Page 1-6

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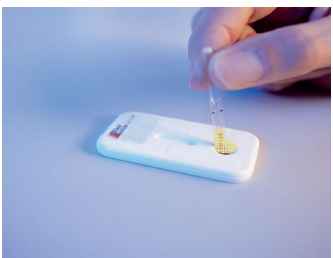
Referenzen

1. Greinacher A. Heparin-induced thrombocytopenia. J Thromb Haemost. 2009; 7 Suppl 1:9-12. Review.
2. Wilke T, Tesch S, Scholz A, Kohlmann T, Greinacher A. The costs of heparin-induced thrombocytopenia: a patient-based cost of illness analysis. J Thromb Haemost. 2009;7(5):766-73
3. Bakchoul T, Giptner A, Najaoui A, Bein G, Santoso S, Sachs UJ. Prospective evaluation of PF4/heparin immunoassays for the diagnosis of heparin-induced thrombocytopenia. J Thromb Haemost. 2009;7 (8):1260-5.
4. Sachs UJ, Manuskript in Vorbereitung
5. Warkentin TE, Greinacher A, Koster A, Lincoff AM; American College of Chest Physicians. Treatment and prevention of heparin-induced thrombocytopenia: American College of Chest Physicians Evidence-Based Clinical Practice Guidelines (8th Edition). Chest. 2008 Jun;133(6 Suppl):340S-380S.

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